

CareBidet Installation, Cleaning and Maintenance



Overview

The CareBidet® is manufactured by Curaco Inc. and designed for both clinical and home environments. This guide summarises what to expect during installation, day to day cleaning, and periodic maintenance.

WeissenBlu provides full onboarding, training and ongoing service support with every unit.



Installation at a Glance

- **Placement:** The main unit sits bedside on a stable surface, connected to standard mains power.
- **Bed compatibility:** Works with most standard care and hospital beds, and the user remains in bed throughout care.
- **Wearable receiver:** A soft, disposable lined receiver is positioned between the legs and connects to the main unit via a sealed line.
- **Water and drainage:** The system uses its own water reservoir, and waste is captured in a sealed, removable container.
- **Setup time:** Typically 30 to 45 minutes with a WeissenBlu clinician on site for first setup and training.

Daily Cleaning Routine

Frequency	Task	Approx. Time
Every use	Automatic wash and dry cycle triggered by sensor	Automated
Daily	Empty and rinse the sealed waste container	2 to 3 min
Daily	Wipe external surfaces with mild disinfectant	1 to 2 min
Daily	Refill the clean water reservoir	1 min
Weekly	Inspect nozzle, hoses and receiver seals	5 min
Monthly	Filter check and replacement per user manual	10 min

Safety and Maintenance Principles

- **Temperature limits:** Water and air temperatures are factory limited to protect fragile skin.
- **Sealed waste path:** Reduces odour, cross contamination and carer biohazard exposure.
- Never bypass the manufacturer's cleaning intervals, as this preserves warranty and hygiene standards.
- Report any unusual noise, alarms, or performance changes to WeissenBlu promptly.

WeissenBlu Support: Every rental and purchase includes on site setup, carer training, a full user manual, and access to our clinical support line: 1300 297 752 | info@weissenblu.com.au | weissenblu.com.au